

# WillowWood One® TF System

## Patient Criteria

The Patient criteria for using the WillowWood One System in a transfemoral application are as follows:



### Indicators

Patients who meet the following criteria should be considered for this system:

- Patients who are transfemoral amputees
- Patients who comment about lack of suspension or feeling insecure in their current prosthesis
- Patients who desire improved linkage from higher prosthetic socket trim lines
- Patients with active lifestyles who could benefit from limb volume stabilization and a temperature-controlled prosthetic environment
- Patients who comment about excessive sweating on the residual limb
- Patients who potentially would benefit from better limb health
- Patients who exhibit high cognitive function and are “in tune” with prosthetic fit
- Patients with reasonable hand dexterity
- Community ambulators
- Patients who are technologically savvy, are willing to incorporate electronic devices into their lifestyle, and do not mind using prosthetic components with batteries that must be charged
- Patients with redundant tissue



### Contraindicators

Patients with any of the following conditions may not be considered for this system:

- Patients who are transtibial amputees
- Patients with a residual limb length less than 7 inches (178 mm) or a femoral length less than 6 inches (155 mm)
- Patients with bulbous residual limbs
- Patients with deep scarring or deep invaginations could be a concern due to lack of contact from an OTS gel interface. These patients might be better served by a custom fabricated gel interface.
- Patients with open wounds
- Patients who lack the cognitive ability to understand the donning and doffing technique of the system as well as an understanding the function of the vacuum system
- Patients who lack the ability to don and doff a gel interface or the ability to properly don and doff the system
- Patients who have poor residual limb hygiene or who are unwilling or unable to comply with the cleaning and disinfecting instructions for the liner

**WillowWood**®

The Ohio Willow Wood Company  
15441 Scioto Darby Road  
Mt. Sterling, OH 43143  
phone 740.869.3377 / 800.848.4930  
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V.  
Keizersgracht 62/64  
1015 CS Amsterdam  
The Netherlands

